



Clear acrylic – digitally enhanced for clarity

BIO-MED DEVICES

ZEE-FRAME

Instructions for use:

1. Interlock to the mattress edge at the head or the side of the patient. For straight through type patient wyes, two Zee-Frames, one on each side, may be used to support the inspiratory and expiratory tubes.
2. To adjust the height and angle of the Zee-Frame to accommodate different sizes and positions of patients, loosen the knob, set the desired position and then, while holding the frame in position, tighten the knob.
3. Insert the tubing in the appropriate slot.
4. Connect the patient to the breathing circuit and make any final adjustments.
5. To clean, use soap or mild non-abrasive detergent. Do not use solvents that may damage the acrylic.
6. **DO NOT HEAT STERILIZE.** The Zee-Frame may be cold sterilized, gas sterilized, or pasteurized.

Specifications:

Catalog #: 2100
Material: clear acrylic
Height: variable from 5 ½" to 8 ½"
Base: 7" x 4 ½"
Mattress: up to 1 ½" thick
Tubing size: 3/8", ½", 1", 9mm, 10mm
Shipping Wt.: 1 lb.

Warranty:

One year against defects in materials and workmanship.



Bio-Med Devices, Inc.
61 Soundview Road, Guilford, CT 06437 U.S.A.
203-458-0202, Fax 203-458-0440
www.biomeddevices.com